



INSTALLATION INSTRUCTIONS

NEWTON SANITIZING MULTI-PURPOSE POUCH

INSTRUCTIONS FOR HELIO C2, HELIO A7, HELIO A6, MOVE* AND HELIO KIDS*

Material(s) required:



Tool(s) required:

No tools required

Installation steps:

1. Begin by removing the footrest on the desired side.
2. On the back side of the pouch, you will find a vertical Velcro strip with four holes and a horizontal Velcro strip with two holes. The positioning straps can be fed through these holes to secure the pouch to the chair.

Feed straps through the top and bottom holes along the Velcro strip.



3. Then, feed the third strap through the left or right hole along the horizontal strip.
 - a. If you are installing the pouch on the left footrest, feed the strap through the right hole.



- b. If you are installing on the right footrest, feed through the left.



4. Pull the three straps taut until the rings are close to the Velcro strips.



5. Holding the pouch up to the footrest, as shown, feed each strap around the tube and into the ring. Cinch and press each strap back down onto itself.



6. Reattach the footrest to the wheelchair.

* MOVE and HELIO KIDS models are not available in the USA.



INSTALLATION INSTRUCTIONS **NEWTON SANITIZING MULTI-PURPOSE POUCH**

INSTRUCTIONS FOR APEX AND VELOCE

Material(s) required:



Tool(s) required:

4 mm hex key

Installation steps:

1. On the back of the pouch, you will find a vertical Velcro strip with four holes. At the bottom of the pouch, there is an eyelet.
4. Feed the screw through the eyelet into the large hole on the outside of the frame saver. Tighten with the 4 mm hex key.

2. Feed the red and black strap through the top hole. Pull the strap taut so the ring sits along the Velcro strip, as shown.



3. Remove the screw from the frame saver using the 4mm hex key. Insert the frame saver at the base of the impact guard with its largest hole on the outside of the chair, as shown.



5. Position the pouch in the upright position. Feed the strap around the impact guard and through the ring. Cinch and press the strap back down onto itself.

