





INSTRUCTION SHEET

NEWTON AIR GRIP HANDRIMS CLEANING

-  • This instruction applies to all Newton Air Grip handrims, regardless of size or color.
-  • **DO NOT USE** rubbing alcohol, paint thinner, strong or abrasive cleaning product, vinegar, a pressure cleaner paper towel or soft tissue towel.

Material(s) required:

- Warm water
- Dishwashing soap
- 2 microfiber rags

Tool(s) required:

- None

CLEANING



1. Put 1 rag in warm water and wring carefully. Keep this rag clean.
2. Mix approximately 500 ml of warm water and 5 ml of dishwashing soap.
3. Put the other microfiber rag in the soapy water and wring carefully.
4. With the soapy rag, clean the rims.
5. With the clean rag, wipe off the excess of soap and water.
6. Allow to air dry.