



INSTRUCTION SHEET

HOW TO TAKE MEASUREMENTS OF A RIGID WHEELCHAIR (APEX A-APEX C-APEX P)

Material(s) required:

- None


Tool(s) required:

- Measuring tape
- Digital protractor

1. **SEATH WIDTH** : From the outside of the frame.



3. **FRAME LENGTH** : From the back end of the frame tube to the beginning of the bend of the footrest.

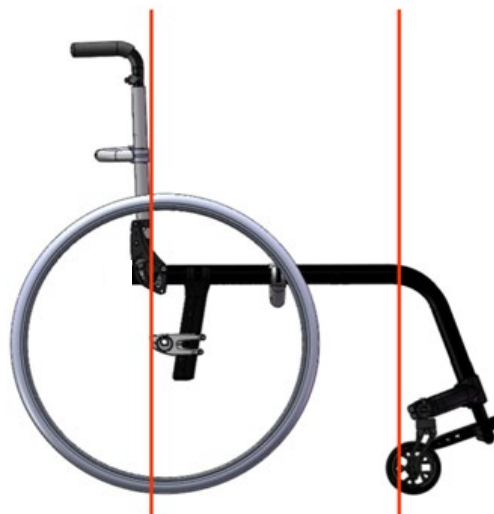

2. **SEAT DEPTH** : From the front of the back cane to the front of the front of the seat upholstery.



4. **BACK CANE ANGLE** : From a level floor to the back cane.



5. **FOOTREST ANGLE** : From the level floor to the front-end tubing.



7. **SEAT TO FOOTREST LENGTH** : From the front of the seat upholstery to the footplate support tube.



6. **FOOTPLATE ANGLE** : From the footplate to the hanger.



8. **ARMREST HEIGHT** : From the top of the seat frame to the top of the armrest pad.



9. **BACK CANE HEIGHT** : From the top of the seat frame to the start of the bend of the push handle OR at the top of the cap.

