



INSTRUCTION SHEET

POSITIONING BELT INSTALLATION/ADJUSTMENT



- The positioning belts in this instruction are NOT RESNA WC-4:2017 section 19 and ISO 7176-19:2019 approved. It shall not in any way be used as security belt for transportation in a motor vehicle.



- Before the beginning of the installation, make sure to have all needed pieces. The pieces must be ordered by your representative or by the customer service from Motion Composites. Have your serial number on hand to facilitate the order.


Matériel(s) requis:

- 1 Positioning belt (auto/aircraft buckle type or Velcro type)

Outil(s) requis:


- 2 clamps #60500
- 2 screws #90665
- 4 mm Allen key

INSTALL/TAKE OFF THE POSITIONING BELT

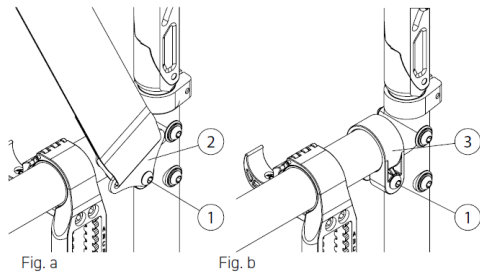

1. Take off the screw (1) to detach the anchor of the positioning belt (Fig. a).
2. Reinstall the screw (1) directly at the same place of the anchor (3) of the (Fig. b).
3. Repeat the 1 and 2 steps on the other side or installation.
4. Screw completely with the Allen key both sides.

AUTO BUCKLE AND AIRCRAFT BUCKLE POSITIONING BELT ADJUSTMENT

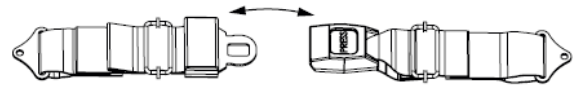

1. To fasten, insert the clip into the buckle until it snaps.
2. To unfasten, lift the flap on the aircraft buckle belt **OR** push the button on the auto buckle belt.
3. To adjust the seat belt, pull each strap towards the opposite side until you get the desired tension and keep the buckle centered.

VELCRO POSITIONING BELT ADJUSTMENT

1. To fasten, insert the long side of the belt into the buckle on the other side.
2. Apply pressure on the belt for a firm grip of the Velcro.