



INSTRUCTION SHEET

HANDRIM REPLACEMENT (WITHOUT TAKING OFF THE TIRE)

- i** • Compatible with the Newton One wheels.
- Before beginning assembly, make sure you have all the necessary parts. These parts must be ordered through your representative or through the Moton Composites customer service. Make sure to have the serial number of your wheelchair to make ordering easier.

Material(s) required:

- 1 air grip handrim

Tool(s) required:

- 4 mm Allen key

1. Place the wheel on a surface to be sure to not scratching it.
2. Unscrew all the screws of the handrim.



3. Change the handrim and align the screw holes with those of the wheel.
4. Screw and tight all the screws.