



INSTRUCTION SHEET REFLECTIVE STICKERS INSTALLATION

- i** • Make sure the surface of the frame used for the stickers is clean before applying them.

COMPATIBLE WITH ALL MODELS

Material(s) required:

- 4 white stickers (#0051328A-00)
- 2 red stickers (0051328A-01)

Tool(s) required:

- None

INSTALLING THE WHITE STICKERS ON THE FOOTREST

1. A sticker must be affixed **to the front of the footrest on both sides.**



2. For **HELIO** models, a sticker must be affixed **to the top of the frame curve on both sides.**



INSTALLING THE WHITE STICKERS ON THE SIDES OF THE FRAME

1. For **APEX** models, a sticker must be affixed **as high as possible outside the frame on both sides.**



INSTALLING THE RED STICKERS

1. A sticker must be affixed **behind the wheelchair frame on both sides.**

