



INSTRUCTION SHEET

SPOKE GUARDS INSTALLATION

- i** Before beginning assembly, make sure you have all the necessary parts. These parts must be ordered through your representative or through the Moton Composites customer service. Make sure to have the serial number of your wheelchair to make ordering easier.

COMPATIBLE WITH ALL MODELS

Material(s) required:	Tool(s) required:
<ul style="list-style-type: none"> Spoke guards assembling kit (#9001629_9001632) 	<ul style="list-style-type: none"> None

SPOKE GUARDS INSTALLATION

9001629_9001632
Spoke Guards

ITEM NO.	PARTNO	DESCRIPTION	9001629 /QTY.	9001630 /QTY.	9001631 /QTY.	9001632 /QTY.
1	51019	SPOKE GUARD 20in	1	-	-	-
2	51020	SPOKE GUARD 22in	-	1	-	-
3	51021	SPOKE GUARD 24in	-	-	1	-
4	51025	SPOKE GUARD 26in	-	-	-	1
5	51034	SPOKE SHIELD CLIP	6	6	6	6
6	51033	SPOKE SHIELD CLIP WASHER	-	-	6	-

DETAIL A
SCALE 5 : 8

ONLY ON THE 24" VERSION OF THE SPOKE GUARD

DETAIL B
SCALE 5 : 8

MAKE SURE TO HAVE YOUR SERIAL NUMBER NEARBY WHEN ORDERING PARTS FOR YOUR MOTON COMPOSITES WHEELCHAIR

- Fix the spoke guard on the wheel by hooking the clips on the spokes of the wheel.

- Add a washer to each clip for 24 inches wheels.

