



INSTRUCTION SHEET
ADJUSTABLE TENSION BACKREST INSTALLATION (APEX A/APEX C)

- i** Before beginning assembly, make sure you have all the necessary parts. These parts must be ordered through your representative or through the Moton Composites customer service. Make sure to have the serial number of your wheelchair to make ordering easier.


Material(s) required:

- Rigid frame seat sling with hook (#9008165A_9008229A)

Tool(s) required:

- 4 mm Allen key

9008165A_9008229A
V2 ADJ TENSION ACTIVE-MESH

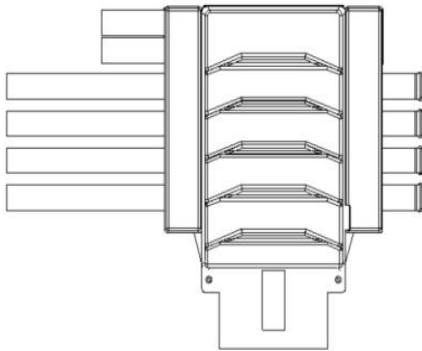


All Back Upholstery comes with an appropriate number of straps

Back Upholstery Replacement Parts

Top Straps		Bottom Straps		Upholstery Tab
<i>1 per Upholstery</i>		<i>Qty depend Upholstery height</i>		<i>1 per Upholstery</i>
Part#	Backrest Width in (cm)	Part#	Backrest Width in (cm)	Part#
0063110A-00	12-13 (30.5-33)	0063117A-00	12-13 (30.5-33)	0063124A-00
0063111A-00	14-15 (35.6-38.1)	0063118A-00	14-15 (35.6-38.1)	
0063112A-00	16-17 (40.6-43.2)	0063119A-00	16-17 (40.6-43.2)	
0063113A-00	18-19 (45.7-48.3)	0063120A-00	18-19 (45.7-48.3)	
0063114A-00	20-21 (50.8-53.3)	0063121A-00	20-21 (50.8-53.3)	
0063115A-00	22 (55.9)	0063122A-00	22 (55.9)	

V2 ADJUSTABLE TENSION BACK UPHOLSTERY ACTIVE-MESH		LENGTH in inches (cm)				
		9-12 (22.9-30.5)	12-15 (30.5-38.1)	15-18 (38.1-45.7)	18-21 (45.7-53.3)	21-24 (53.3-60.9)
# OF STRAPS INCLUDED		1 Strap	2 Straps	3 Straps	4 Straps	5 Straps
WIDTH in inches (cm)	12 (30.5)	9008165A-00	9008166A-00	9008167A-00	9008168A-00	9008169A-00
	13 (33.0)	9008170A-00	9008171A-00	9008172A-00	9008173A-00	9008174A-00
	14 (35.6)	9008175A-00	9008176A-00	9008177A-00	9008178A-00	9008179A-00
	15 (38.1)	9008180A-00	9008181A-00	9008182A-00	9008183A-00	9008184A-00
	16 (40.6)	9008185A-00	9008186A-00	9008187A-00	9008188A-00	9008189A-00
	17 (43.2)	9008190A-00	9008191A-00	9008192A-00	9008193A-00	9008194A-00
	18 (45.7)	9008195A-00	9008196A-00	9008197A-00	9008198A-00	9008199A-00
	19 (48.3)	9008200A-00	9008201A-00	9008202A-00	9008203A-00	9008204A-00
	20 (50.8)	9008205A-00	9008206A-00	9008207A-00	9008208A-00	9008209A-00
	21 (53.3)	9008210A-00	9008211A-00	9008212A-00	9008213A-00	9008214A-00
22 (55.9)	9008215A-00	9008216A-00	9008217A-00	9008218A-00	9008219A-00	

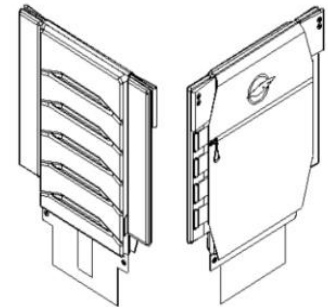

ex: 9008165A-00 (Upholstery w/ straps)
 ex: 0062165B-00 (Upholstery only)

Instance	# of Velcro Loop Patches
-00	None
-01	1 Velcro Patch
-02	2 Velcro Patches
-03	3 Velcro Patches
-04	4 Velcro Patches
-05	5 Velcro Patches
-06	6 Velcro Patches



INSTRUCTION SHEET ADJUSTABLE TENSION BACKREST INSTALLATION (APEX A/APEX C)

INSTALLATION

The number of backrest straps depend on its format :

- 9 à 12 inches : 1 strap
- 12 à 15 inches : 2 straps
- 15 à 18 inches : 3 straps
- 18 à 21 inches : 4 straps

* This instruction was made with a 4-strap backrest but the procedure is the same for all backrests.


1. Unscrew and remove the push handles using the 4 mm Allen key (if necessary).



2. For each strap, you must insert the strap into the eyelet, keeping the velcro part on the outside.




3. Insert the straps on the left back cane. Make sure the velcro is facing you.

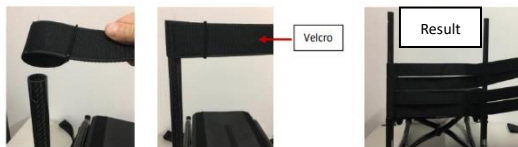

4. Adjust the double strap to the width of the wheelchair.



5. Install the push handles and screw them to the double strap on each side.

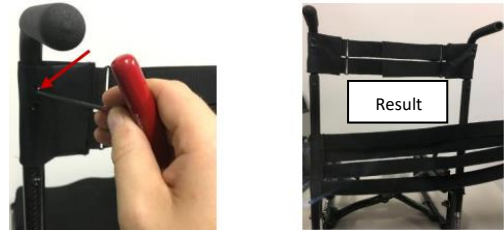


6. Lift the folding part of the backrest on each side.



7. Install the backrest so that the velcros of the backrest stick to the velcros of the double strap.



8. Insert the straps in the loops on the left of the backrest and attach them to the right back cane.

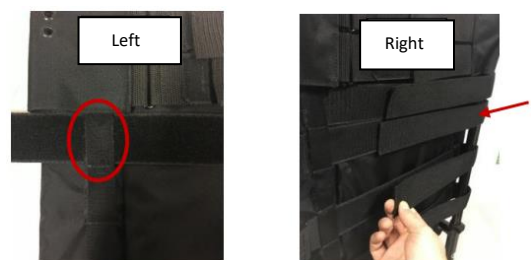



INSTRUCTION SHEET

ADJUSTABLE TENSION BACKREST INSTALLATION (APEX A/APEX C)

9. Tighten the straps on the velcro behind the wheelchair as desired.

! BE CAREFULL NOT TO TIGHTEN THE STRAPS TOO MUCH: RISK OF DAMAGE TO THE BACK CANES.

10. Fold down the back panel.

