



## INSTRUCTION SHEET SLIP-ON BACKREST TIE-WRAPS INSTALLATION

- i** Before beginning assembly, make sure you have all the necessary parts. These parts must be ordered through your representative or through the Moton Composites customer service. Make sure to have the serial number of your wheelchair to make ordering easier.

### Material(s) required:

- 2 X tie-wraps (#0011489A-00)

### Tool(s) required:


- None

## INSTALLATION

1. **SLIGHTLY** tighten the tie-wraps on each side **ALTERNATELY** left and right to keep the back upholstery straight until the desired tension.



- Tighten, with the same tension, the left and right tie-wrap.

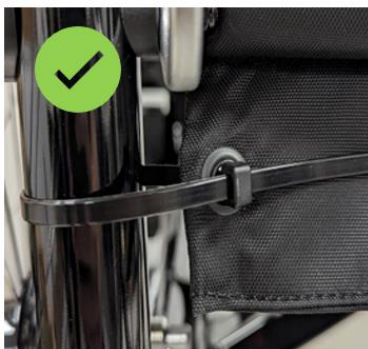

2. Cut the excess **as close as possible** to the fastener to void rubbing on the upholstery.

