



INSTRUCTION SHEET

SEAT SLING INSTALLATION (APEX A - APEX C – APEX P)



- As many straps as possible should always be installed on the wheelchair and keep the remaining ones in case of a future modification of the wheelchair (see tables below).
- IF THE MINIMUM NUMBER OF STRAPS CANNOT BE INSTALLED, REMOVE ONE OR MORE ACCESSORIES.



- The 10” and 11” deep seat slings are incompatible with T armrests and side guards.
- Before beginning assembly, make sure you have all the necessary parts. These parts must be ordered through your representative or through the Moton Composites customer service. Make sure to have the serial number of your wheelchair to make ordering easier.

Seat depth	Number of straps delivered to the BOM
10po	4
11po	4
12po	4
13po	5
14po	5
15po	6
16po	7
17po	8
18po	9
19po	10
20po	11

Seat depth	Minimum number of straps installed
10po	3
11po	3
12po	3
13po	4
14po	4
15po	5
16po	6
17po	7
18po	7
19po	8
20po	9

Material(s) required:

- 1 X Seat strap kit 1” – assy (#9008113A)

Tool(s) required:

- None

INSTALLATION

1. Insert the strap into the clamps on each side of the Velcro.



- It is important to ensure that the stap is inserted correctly into the slots of the clamp.



2. Install the clamps (with straps) on the wheelchair H-Block wherever space permits.


3. Pass the strap through the eyelet on the opposite side.





INSTRUCTION SHEET
SEAT SLING INSTALLATION (APEX A - APEX C – APEX P)

4. Pass the strap through the eyelet on the opposite side.

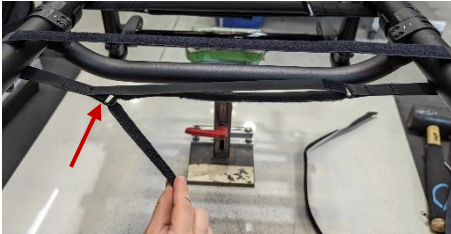

5. Adjust to the desired tension and press the Velcro.



6. Apply seat sling to Velcro.