



INSTRUCTION SHEET

HIGH MOUNT FOOTPLATES

1

- i** Before beginning assembly, make sure you have all the necessary parts. These parts must be ordered through your representative or through the Motion Composites customer service. Make sure to have the serial number of your wheelchair to make ordering easier.

Material(s) required:

Tool(s) required:

➤ Footplates:

- 0031025-00 • COMPOSITE FOOTPLATE 6x6in - ASSY
- 0031026-00 • COMPOSITE FOOTPLATE 7x6in - ASSY
- 0031027-00 • COMPOSITE FOOTPLATE 8x6in - ASSY
- 0031589-00 • ADJ NEWTON COMP FOOTPLATE 5.5x8in, RIGHT - ASSY
- 0031590-00 • ADJ NEWTON COMP FOOTPLATE 5.5x8in, LEFT - ASSY
- 0031591-00 • ADJ NEWTON COMP FOOTPLATE 6.5x8in, RIGHT - ASSY
- 0031592-00 • ADJ NEWTON COMP FOOTPLATE 6.5x8in, LEFT - ASSY
- 0031593-00 • ADJ NEWTON COMP FOOTPLATE 7.5x8in, RIGHT - ASSY
- 0031594-00 • ADJ NEWTON COMP FOOTPLATE 7.5x8in, LEFT - ASSY
- 0031595-00 • ADJ NEWTON ALU FOOTPLATE 4.5x8in, RIGHT - ASSY
- 0031596-00 • ADJ NEWTON ALU FOOTPLATE 4.5x8in, LEFT - ASSY
- 0031597-00 • ADJ NEWTON ALU FOOTPLATE 5.5x8in, RIGHT - ASSY
- 0031598-00 • ADJ NEWTON ALU FOOTPLATE 5.5x8in, LEFT - ASSY
- 0031599-00 • ADJ NEWTON ALU FOOTPLATE 6.5x8in, RIGHT - ASSY
- 0031600-00 • ADJ NEWTON ALU FOOTPLATE 6.5x8in, LEFT - ASSY
- 0031601-00 • ADJ NEWTON ALU FOOTPLATE 7.5x8in, RIGHT - ASSY
- 0031602-00 • ADJ NEWTON ALU FOOTPLATE 7.5x8in, LEFT - ASSY
- 0031603-00 • ADJ NEWTON ALU FOOTPLATE 4.5x10in, RIGHT - ASSY
- 0031604-00 • ADJ NEWTON ALU FOOTPLATE 4.5x10in, LEFT - ASSY
- 0031605-00 • ADJ NEWTON ALU FOOTPLATE 5.5x10in, RIGHT - ASSY
- 0031606-00 • ADJ NEWTON ALU FOOTPLATE 5.5x10in, LEFT - ASSY
- 0031613-00 • ADJ NEWTON ALU FOOTPLATE 6.5x10in, RIGHT - ASSY
- 0031623-00 • ADJ NEWTON ALU FOOTPLATE 6.5x10in, LEFT - ASSY
- 0031633-00 • ADJ NEWTON ALU FOOTPLATE 7.5x10in, RIGHT - ASSY
- 0031670-00 • ADJ NEWTON CARBON FOOTPLATE 5.5x8in, RIGHT - ASSY
- 0031608-00 • ADJ NEWTON CARBON FOOTPLATE 5.5x8in, LEFT - ASSY
- 0031609-00 • ADJ NEWTON CARBON FOOTPLATE 6.5x8in, RIGHT - ASSY
- 0031610-00 • ADJ NEWTON CARBON FOOTPLATE 6.5x8in, LEFT - ASSY
- 0031611-00 • ADJ NEWTON CARBON FOOTPLATE 7.5x8in, RIGHT - ASSY
- 0031612-00 • ADJ NEWTON CARBON FOOTPLATE 7.5x8in, LEFT - ASSY

➤ Clamps: 41703 x4


➤ Extensions : 31638-00 x 2

• For the clamps:

- 5mm Allen Key
- 12Nm adjusted torque wrench
- Blue threadlocker


• For the footplates:

- 4mm Allen Key
- Ratchet with 10mm socket

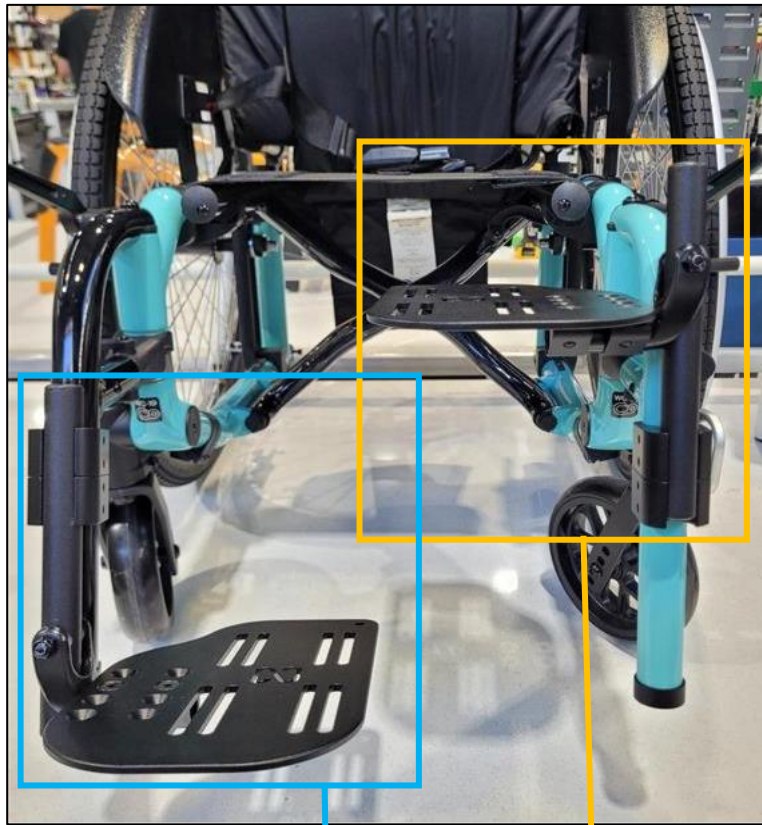


11" and lower assembly

10.5" and higher assembly

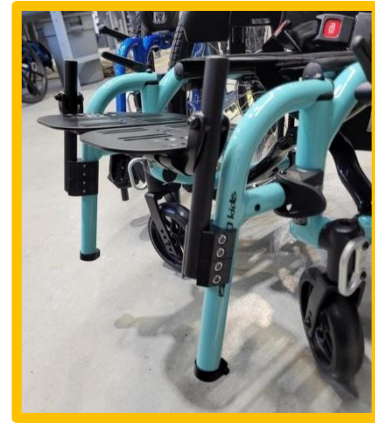


Angle adjustable footplates :

1. Remove the hardware from the footplate assembly.
2. Insert the extension and install the hardware with the head of the screw at the back of the footplate.
3. Tighten the assembly with a 4mm Allen key and a ratchet with a 10mm socket.

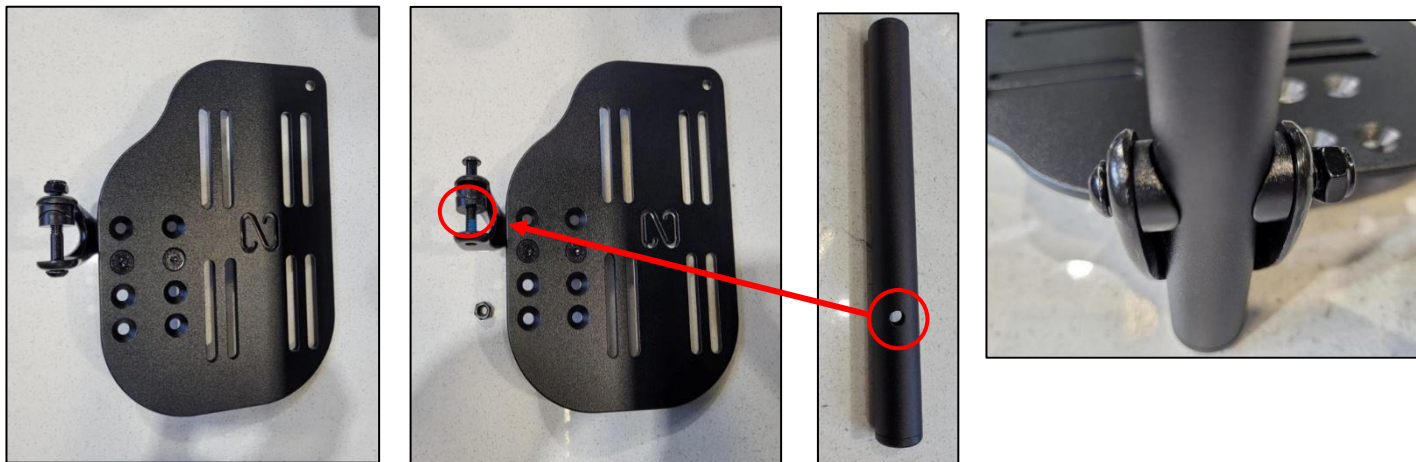


Standards footplates :

1. Remove the hardware from the footplate.
2. Insert the extension and put back the hardware as it is.
3. Tighten the assembly with a 4mm Allen key.

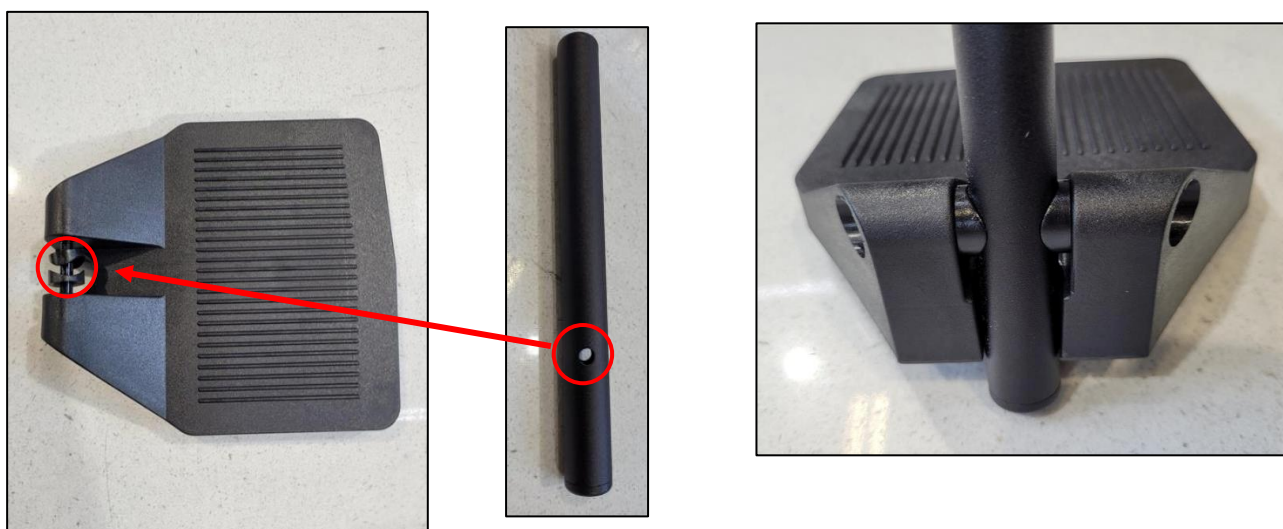

Clamps Installation :

1. Remove the screws from the clamps using a 5mm Allen key and put a drop of blue threadlocker on them.
2. Put back the screws on the clamps.

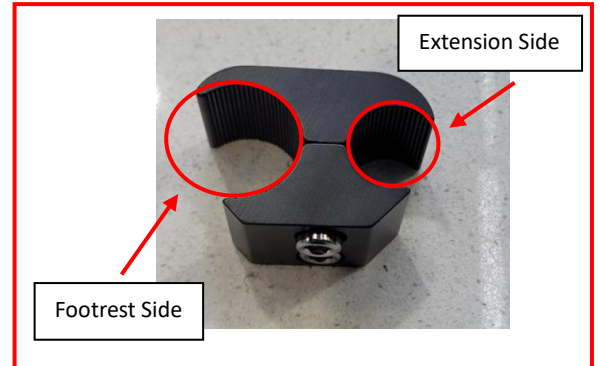

3. Install the clamps on the footrest (screws facing the outside of the footrest).
4. Holding the clamps , insert the extension.
5. Adjust the extension at the desired height .
6. Screw the clamps with a 5mm Allen Key .
7. Torque the screws at 12 Nm.

**** For a better range of adjustment , clamps should be stacked and not separated ****

