



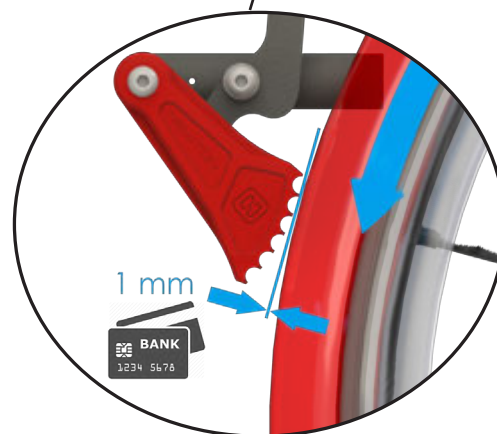

NEWTON GRADE AID WHEEL LOCK

1. **Open** mode - 2. **Grade aid** mode - 3. **Lock on** mode

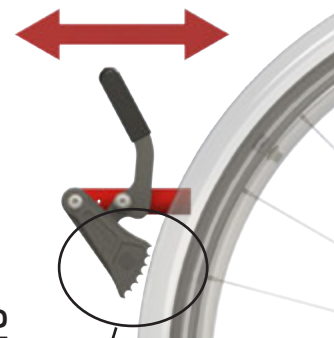

3 positions Newton Grade Aid Wheel Lock

INSTALLATION

1. Place the wheel lock in the open mode position (Setting 1).
2. Adjust the distance of the brake in order to leave the space of a bank card or approx. 1mm between the teeth and lock the tread of the tire. Rotate the tire to verify consistent spacing between the brake teeth and the entire circumference of the tire.
3. Tighten the screws on the brake clamp.

CODE :

41 035 a.0 RIGHT GRADE AID WHEEL LOCK - ASSEMBLY

41 036 a.0 LEFT GRADE AID WHEEL LOCK - ASSEMBLY