



INSTRUCTION SHEET

SEAT SLING INSTALLATION WITH 1" STRAPS (APEX A-C-P)



- As many straps as possible should always be installed on the wheelchair and keep the remaining ones in case of a future modification of the wheelchair (see tables below).
- IF THE MINIMUM NUMBER OF STRAPS CANNOT BE INSTALLED, REMOVE ONE OR MORE ACCESSORIES.



- The 10" and 11" deep seat slings are incompatible with T armrests and side guards.
- Before beginning assembly, make sure you have all the necessary parts. These parts must be ordered through your representative or through the Moton Composites customer service. Make sure to have the serial number of your wheelchair to make ordering easier.

Seat depth	Number of straps supplied when built
10" / 25.4cm	4
11" / 27.94cm	4
12" / 30.48cm	4
13" / 33.02cm	5
14" / 35.56cm	5
15" / 38.1cm	6
16" / 40.64cm	7
17" / 43.18cm	8
18" / 45.72cm	9
19" / 48.26cm	10
20" / 50.8cm	11

Seat depth	Minimal number of straps installed
10" / 25.4cm	3
11" / 27.94cm	3
12" / 30.48cm	3
13" / 33.02cm	4
14" / 35.56cm	4
15" / 38.1cm	5
16" / 40.64cm	6
17" / 43.18cm	7
18" / 45.72cm	7
19" / 48.26cm	8
20" / 50.8cm	9



INSTRUCTION SHEET

SEAT SLING INSTALLATION WITH 1" STRAPS (APEX A-C-P)

Material(s) required:

- 1 X Seat strap kit 1" – assy (#9008113A)

Tool(s) required:

- None

INSTALLATION

1. New straps with neoprene for Apex A-C-P.




2. Install the straps on the wheelchair frame at all locations where the required space allows, with the neoprene sections resting against the frame.



3. Pass the strap through the eyelet on the opposite side.



4. Pass the strap through the eyelet on the opposite side.

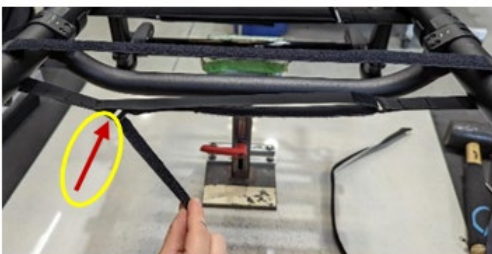

5. Adjust the strap to the desired tension and press down on the Velcro.



6. Attach the seat upholstery to the Velcro.